

hygitech

Absolute Hygiene



DENTO-VIRACTIS



HANDS



INSTRUMENTS



SURFACES



ASPIRATION



BURS

A complete range of innovative disinfectants

DENTO-VIRACTIS



HYGITECH is a partner of health professionals in the fight against nosocomial infections.

DENTO-VIRACTIS represents a full range of disinfection products developed by the French laboratory Hygitech / C.F.P.M.

DENTO-VIRACTIS range was developed by our team of renowned researchers and chemists. It's the result of intensive and continuous research in a specialised laboratory.

DENTO-VIRACTIS provides a full range of very high quality products pursuant to CE conventions and manufactured in certified plants.

These products are tested in our COFRAC-approved partner laboratory and meet the more stringent standards.

The DENTO-VIRACTIS range consists of pre-disinfectants and final disinfectants for instruments as well as cleansers and deodorisers for surfaces and suction systems or hands disinfection. These products are very pleasant and easy to use.

DENTO-VIRACTIS guarantees an absolute hygiene and you can be sure of effectively combating a range of infection hazards, safe in the knowledge that you are also getting the best in value, from a prominent name in European dentistry.

hygitech

25, rue Bleue
75009 Paris - FRANCE
Tel.: +33 (0)1 48 01 32 88
Fax.: +33 (0)1 48 01 32 80
e-mail: sales@hygitech.fr
Website: www.hygitech.com





ROTARY INSTRUMENTS DENTO-VIRACTIS 57

BURS DISINFECTANT

- All material compatibility
- Ready to use solution
- Rapid action: bactericidal, fungicidal & virucidal action in 15 minutes only



INDICATIONS

Cleaning and disinfecting of cutters and rotary instruments prior to sterilization.
Just after burs and rotary instruments use.

INGREDIENTS

Propanol, Potassium hydroxide, fatty acids. Aldehyde free.

INSTRUCTIONS FOR USE

- Submerge the devices in the **DENTO-VIRACTIS 57** solution for 15 minutes.
- Rinse. The solution must be changed every day.

PROPERTIES

Bactericidal: EN 1040, AFNOR NFT 72-171/301, AFNOR NFT 72-190.
Fungicidal: EN 1275.
Virucidal: AFNOR NFT 72-180, tested on HBV.

DESIGNATION

2 L bottle
5 L bottle

REF.

DV - 571
DV - 572



SURFACES DENTO-VIRACTIS 77

CLEANING AND DISINFECTION OF SURFACES

- Cleans and disinfects at the same time
- Very pleasant grapefruit scent
- Dries rapidly and without trace
- Ready to use solution



INDICATIONS

Hydro-alcoholic spray without colorant and aldehyde specially formulated for cleaning and disinfection of all surfaces (units, chairs, table...) equipment and non sterilisable or disinfectable medical devices with standardized methods.

INSTRUCTIONS FOR USE

- 1- Cleaning:
spray **DENTO-VIRACTIS 77** on a disposable non-woven surface. Clean by moistening the entire surface treated.
- 2- Disinfection:
spray **DENTO-VIRACTIS 77** on the surface. Allow the anti-microbe film to dry on the surface.

PROPERTIES

Bactericidal : NF EN 1040, AFNOR T 72-190, AFNOR XPT 72-301 (BK)
Fungicidal : NF EN 1275 (Candida albicans), AFNOR 72-201.
Virucidal : active on poliovirus AFNOR NFT 72-180. Active on Rotavirus, Adenovirus, Orthopoxvirus , HIV, HBV

DESIGNATION

1 L bottle with spray
5 L bottle

REF.

DV - 771
DV - 772

SURFACES DENTO-VIRACTIS 77

WIPES FOR QUICK DISINFECTION

- 100% compatible with DENTO-VIRACTIS 77 Spray
- Very pleasant grapefruit fragrance
- Lid with Individually wrapped box to avoid any liquid leaks
- Easy to tear off wipes



INDICATIONS

Disposable moist wipes for quick cleaning and disinfecting surfaces and medical devices between two patients.

INSTRUCTIONS FOR USE

- 1- Pull out a wipe. Tear it off cleanly.
 - 2- Close the box tightly.
 - 3- Wipe the surfaces or equipment.
- Do not rinse, leave to dry.

PROPERTIES

DENTO-VIRACTIS 77 wipes are soaked in a disinfectant solution with the following activities:

Bactericidal : NF EN 1040, AFNOR T 72-190, AFNOR XPT 72-301 (BK)

Fungicidal : NF EN 1275 (Candida albicans), AFNOR 72-201.

Virucidal : active on poliovirus AFNOR NFT 72-180. Active on Rotavirus, Adenovirus, Orthopoxvirus , HIV, HBV

DESIGNATION REF.

120 wipes

DV - 773



ANTISEPTIC SOAP DENTO-VIRACTIS 33

MILD ANTISEPTIC SOAP

- Clean and disinfect
- Surgical hand-washing
- Broad spectrum of activity
- Suitable for frequent use
- Very pleasant floral scent



INDICATIONS

Combines anti-microbial efficiency with high cutaneous tolerance. A broad spectrum of activity, designed for hygienic or surgical hand-washing for all persons working under aseptic conditions (surgeons, dentists, physicians, nurses, etc.) or entering operating blocks or industrial clean rooms.

INSTRUCTIONS FOR USE

- Antiseptic wash: wet hands and forearms, apply 2 ml of product (the quantity of **DENTO-VIRACTIS 33** obtained by one press on the pump dispenser), scrub for 2 mins, rinse well and dry.
- Surgical wash: after an initial antiseptic wash, scrub nails and repeat antiseptic wash as per protocol.

PROPERTIES

Bactericidal: NF EN 1040, NF EN 1499, T72-301 VS *Salmonella enterica* *listeria monocytogenes*.
Fungicidal: NF T72-201 VS *candida albicans*.

DESIGNATION

REF.

| | |
|--------------------------------|----------|
| 1 L bottle with pump dispenser | DV - 331 |
| 5 L bottle | DV - 332 |
| Pump dispenser for 5 L bottle | DV - 314 |

DISINFECTION LOTION DENTO-VIRACTIS 35

HYDRO-ALCOHOLIC LOTION

- Skin protection that keeps soft hands
- No need to rinse, dry fast
- Wide spectrum of activity
- Very pleasant scent



INDICATIONS

Antiseptic hydro-alcoholic solution designed for the hygienic treatment and surgical disinfection of hands by rubbing.

INSTRUCTIONS FOR USE

- 1- Apply 2 ml of product (the quantity of **DENTO-VIRACTIS 35** obtained by one press on the pump dispenser) after a hygienic or surgical handwash.
- 2- Scrub for at least 30 seconds.
- 3- Allow to dry without rinsing.

PROPERTIES

Bactericidal: NF EN 1040.
Fungicidal: NF EN 1275.
Virucidal: HIV.

DESIGNATION

1 L bottle
5 L bottle
Pump dispenser for 1 L bottle
Pump dispenser for 5 L bottle

REF.

DV - 351
DV - 352
DV - 313
DV - 314



DISINFECTANT
DENTO-VIRACTIS 35
GEL

HYDRO-ALCOHOLIC GEL

- Skin protection that keeps soft hands
- No need to rinse, dry fast
- Wide spectrum of activity
- Very pleasant scent



INDICATIONS

Antiseptic hydro-alcoholic gel designed for the hygienic treatment and surgical disinfection of hands by rubbing.

INSTRUCTIONS FOR USE

- 1- Apply 2 ml of product (the quantity of **DENTO-VIRACTIS 35 Gel** obtained by one press on the pump dispenser) after a hygienic or surgical handwash.
- 2- Scrub for at least 30 seconds.
- 3- Allow to dry without rinsing.

PROPERTIES






Bactericidal: NF EN 1040, NF EN 1500, NF EN 12791.
Fungicidal: NF EN 1275 (Candida albicans).











DESIGNATION

1 L bottle with Pump dispenser

REF.

DV - 354

| | | |
|---|---|--|
|  HANDS | HAND CLEANING DENTO-VIRACTIS 31 | Mild liquid lotion for frequent hand-washing. 1 L and 5 L bottles. |
| | ANTISEPTIC SOAP DENTO-VIRACTIS 33 | Mild antiseptic soap for hand-washing and quick disinfection. 1 L and 5 L bottles. |
| | DISINFECTION LOTION DENTO-VIRACTIS 35 | Hydro-alcoholic solution for quick hand disinfection. 1 L and 5 L bottles. |
| | DISINFECTANT DENTO-VIRACTIS 35 GEL | Hydro-alcoholic gel for quick hand disinfection. 1 L bottle. |
|  INSTRUMENTS | INSTRUMENTS DENTO-VIRACTIS 55 | Pre-disinfectant detergent enzymatic powder for decontamination of dental instruments before autoclave sterilisation or cold chemical disinfection. 2 Kg and 5 Kg buckets. Graduated measuring device supplied. |
| | INSTRUMENTS DENTO-VIRACTIS 55 DOSES | Same product than Dento-Viractis 55 but in water soluble powder doses. One dose of 15 g = a bath of 3 L of ready to use solution. Bucket of 50 doses of 15 g. |
| | INSTRUMENTS DENTO-VIRACTIS 59 | Cleaning and pre-disinfectant liquid for soiled dental devices before sterilisation or chemical disinfection. 1 L bottle concentrated with 20 ml dosing device and 5 L bottle concentrated with 20 ml pump dispenser. |
| | INSTRUMENTS DENTO-VIRACTIS 53 INSTRUMENTS DENTO-VIRACTIS 53 READY TO USE | Cold disinfectant liquid for heat sensitive or very fragile medical devices unsterilisable by traditional methods (autoclave). Use after pre-disinfection or de-proteinising cleaning. 1 L bottle concentrated with 20 ml dosing device and 5 L bottle ready to use solution. |
|  BURS | ROTARY INSTRUMENTS DENTO-VIRACTIS 57 | Cleaning and disinfecting of cutters and rotary instruments prior to sterilisation. 2 L and 5 L bottles. Ready to use solution. |
|  SURFACES | SURFACES DENTO-VIRACTIS 77 | Hydro-alcoholic spray without colorant dye and aldehyde free specially formulated for cleaning and disinfection of all surfaces (units, chairs, table...) equipment and non sterilisable or disinfectable medical devices with standardized methods. 1 L bottle with spray, 5 L bottle and 120 wipes for quick disinfection of surfaces and medical devices. |
| | SURFACES DENTO-VIRACTIS 79 | 100 thick and large wipes for surfaces and medical devices disinfection. |
|  ASPIRATION | ASPIRATION DENTO-VIRACTIS 99 | Deodorising cleaning liquid for treating surgical aspiration systems, spittoons and amalgam collectors. 1 L bottle concentrated with 20 ml dosing device and 5 L bottle concentrated with 20 ml pump dispenser. |

| DENTO-VIRACTIS PRICE LIST | | | | | | | |
|--|---------|--|---|--|---------|---|--------|
| VAT INCLUDED | | | | | | | |
| <u>Dento-Viractis 77 - Απολυμαντικό υγρό επιφανειών</u> | | | | | | | |
| Καθαρίζει και απολυμαίνει ταυτόχρονα. Πολύ ευχάριστο άρωμα γκρέιπφρουτ. Βακτηριοκτόνο, μυκητοκτόνο, ιοκτόνο και φυματοκτόνο. Στεγνώνει γρήγορα και χωρίς ίχνος. Χωρίς αλδεΐδη . Κατάλληλο για όλες τις επιφάνειες. | | | | | | | |
| PRODUCT | PRICE | PRODUCT | PRICE | PRODUCT | PRICE | PRODUCT | PRICE |
|  DV772 - 5L | € 61,58 |  DV77 1L spray | € 22 |  DV773 Wipes | € 12,32 |  DV774 Rwipes | € 9,58 |
| <u>Dento-Viractis DV35 - Απολυμαντικό gel για ΑΗ1-Ν1</u> | | | | | | | |
| Αντισηπτικό υδρο-αλκοολικό gel σχεδιασμένο για την υγιεινή επεξεργασία και η χειρουργική αντισηψία των χεριών με το τρίψιμο. Δημιουργεί ένα λεπτό στρώμα στα χέρια με αποτέλεσμα να μην κάθονται τα υπολείμματα της πούδρας των γαντιών. Μαλακώνει την επιδερμίδα. Ευρύ φάσμα δραστηριοτήτων. Δεν χρειάζεται να ξεπλύνετε, στεγνώνει γρήγορα. Προστασία του δέρματος που κρατά απαλά χέρια. Πολύ ευχάριστο άρωμα | | | | | | | |
| PRODUCT | | PRICE | PRODUCT | | PRICE | | |
|  SANITIZIN G GEL DV35 5L | | € 62,95 |  SANITIZIN G GEL DV35 1L | | € 24,63 | | |
| <u>Dento-Viractis DV57 - Απολυμαντικό φρεζών</u> | | | | | | | |
| Απολυμαντικό διάλυμα έτοιμο προς χρήση για όλα τα περιστροφικά εργαλεία όπως φρέζες, διαμάντια, μικροεργαλεία. Βακτηριοκτόνο, μυκητοκτόνο(HIV, HBV). Χωρίς αλδεΐδες. Βυθίστε τα εργαλεία σας στο διάλυμα για 15' | | | | | | | |
| PRODUCT | | PRICE | PRODUCT | | PRICE | | |
|  BURS DISINFECTI ON DV57 5L | | € 54,74 |  BURS DISINFECT ION DV57 2L | | € 25,32 | | |
| <u>Dento-Viractis DV33 Soap - Αντισηπτικό σαπούνι</u> | | | | | | | |
| Αντισηπτικό σαπούνι με αντιβακτηριδιακές ιδιότητες. απαραίτητο για την σωστή περιποίηση των ταλαιπωρημένων χεριών. | | | | | | | |
| PRODUCT | | PRICE | PRODUCT | | PRICE | | |
|  DISIFECTA NT SOAP DV33 5L | | € 65,95 |  DISIFECTA NT SOAP DV33 1L | | € 26,63 | | |